# DIRECTIONS FOR USE

## Sylc™ SmarTip Single Use

### Prophy Therapy Tips and Adapter

Thank you for purchasing the Sylc™ SmarTip system. Please observe the following information to assure optimal use and safe operation.

#### The Starter Kit includes the following:

- 10 x Sylc™ SmarTip Single Use Prophy Therapy Tips
- 1 x Sylc™ SmarTip Adapter

#### The Refill Kit includes the following:

- 25 x Sylc™ SmarTip Single Use Prophy Therapy Tips

**CAUTION:** Federal law restricts this device to be used by or on the order of a licensed dentist.

**INDICATIONS:** Sylc™ SmarTip is a single-use self-contained air and powder device that is powered by standard house air available at every dental operatory. Sylc™ SmarTip is intended to be used with the Sylc™ SmarTip Adapter.

**DESCRIPTION:** Sylc™ SmarTip is an air polishing device prefilled with Sylc™ cleaning, polishing and desensitization powder designed to provide a prophylaxis treatment for dental enamel and to physically occlude dentin tubules for the management of sensitive teeth. Sylc™ SmarTip is a dry powder application, as such it is recommended to rinse and evacuate after each quadrant for immediate cleaning results or wait until the end of the treatment. It is recommended that the patient not drink or eat at least an hour after treatment. Sylc™ SmarTip is marked with a pink dot.

**INGREDIENTS:** Sylc™ is a dry powder (calcium sodium phosphosilicate), composed of elements that occur naturally in the body's hard tissues (Ca, Na, Si, P, and O). When exposed to an aqueous environment, Sylc™ undergoes a rapid surface reaction, allowing it to physically adhere to exposed dentin and to physically occlude tubules. Within a short period of time, essentially all of the Sylc™ reacts to form hydroxycarbonate apatite (HCA), which is chemically similar to natural tooth mineral. Sylc™ has a neutral flavor.

**Sylc™ SmarTip removes plaque, soft deposits, and surface stains. It is also recommended for the following procedures:**

- Removal of extrinsic stains, e.g. coffee, tea and tobacco
- General cleaning and polishing during the prophylaxis treatment
- Treatment of hypersensitivity
- Surface preparation prior to bonding and sealants
- Before and after scaling and root planning procedure to reduce sensitivity
- Plaque removal prior to fluoride treatments
- Treatment of hypersensitivity pre and post whitening treatments

**CONTRAINDICATIONS:** Sylc™ should not be considered for patients with a known allergy to silica.

**DESCRIPTION:** Sylc™ SmarTip Adapter.

The adapter screws directly to the standard chair side hand piece connector (compatible with 2,3,4,5 and 6 hole adapters). **When operating the adapter, an operating pressure of 35-45 psi (2.4-3.1 bar) is recommended.** Operating at pressures above 45 psi (3.1 bar) may cause excessive powder flow and output resulting in suboptimal performance and may increase the risk of damaging the device. It is recommended to check the pressures with a test gauge prior to set-up and use. Air flow is controlled by the standard chair side foot pedal.

**NOTE:** Water delivery port is sealed off.

**NOTE:** The metal tip has a small mark on its surface designed to regulate the powder flow and improve patient comfort.

**CLEANING OF THE ADAPTER:** Prior to the first use, and after every patient, thoroughly clean and disinfect the surfaces of the Adapter by applying an approved, non-immersion disinfectant solution. Carefully follow all of the directions provided by the manufacturer of the disinfectant. To clean the Adapter, generously spray the disinfectant solution onto a clean cloth and vigorously wipe down all surfaces. Immediately discard the used towel and allow the Adapter to dry after cleaning. It is recommended that the Adapter be sleeved for every use, using a standard plastic infection control sleeve for dental hand pieces.

#### STABILITY:

- Do not use if Sylc™ SmarTip is damaged.
- Store in a DRY place at room temperature, between 15-25 °C and 60-75 °F.

#### HOW SUPPLIED:

- Sylc™ SmarTip is supplied as a starter kit containing 10 prefilled single use tips and one adapter (Item #95101001 - Starter Kit).
- Sylc™ SmarTip is supplied as a refill kit containing 25 prefilled single use tips (Item# 95101501 - Refill Kit).

**WARRANTY:** Adapter has a lifetime warranty, subject to change upon notice. Warranty terms do not apply on the prefilled single use tips.

**TECHNICAL CHANGES:** OSSpray reserves the right to make changes to Sylc™ usage instructions when required in the course of ongoing technical and scientific developments.

## SAFETY PRECAUTIONS:

Read the instructions carefully before using this device. The manufacturer, distributor, or retailer of this product can exercise no control over the use of the Sylc™ SmarTip and adapter.

- Operating at pressures above 45 psi (3.1 bar) may cause excessive powder flow and output resulting in suboptimal performance and may increase the risk of damaging the device.
- The Sylc™ SmarTip is a single use device. Dispose after single patient use regardless of the remaining quantity of powder.
- During the preparation process and application, it is recommended that both clinician and patient wear protective eyewear. If any material gets into the eyes, immediately flush with water.
- Delivering Sylc™ with air polishing equipment at high air pressures (psi), may cause injury to the gum tissues or an emphysema caused by the introduction of air into the soft tissue spaces.
- It is strongly recommended not to direct Sylc™ at the gum tissue or into the gingival sulcus.
- Avoid directing Sylc™ powder toward restorations, crowns and bridgework as this could damage the restoration.
- Avoid directing Sylc™ powder on orthodontic brackets as this could dull the metal.
- DO NOT attempt to sterilize, refill or reuse.
- Mounting and pressurizing the device more than once will degrade device operation.
- Clinicians may practice and experiment on typodonts, pennies, stainless steel, extracted teeth, or glass slides before actual use. These surfaces will simulate both precious and non-precious alloys and porcelain.
- Safety and effectiveness in pregnant women and children have not been established.
- Federal Law restricts this device to sale by or on the order of a licensed dentist.


## USER INSTRUCTIONS

**IMPORTANT - REVIEW SAFETY PRECAUTIONS PRIOR TO USE. Manufacturer is not responsible for damages due to misuse of the product.**

### STEP 1


**Check air pressure:  
35-45 psi (2.4- 3.1 bar).**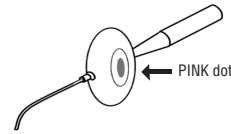

Check that the Sylc™ SmarTip is marked with a PINK dot.

### Removal / Storage

Detaching the Sylc™ SmarTip. Remove tip from connector by pulling back on plastic green collar 1 and at the same time pulling the Sylc™ SmarTip away from the connector. 2 Discard in sharps container.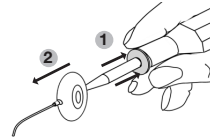


Adapter may be stored at chair side or in a secure place in drawer.

### STEP 2

Prior to inserting Sylc™ SmarTip into adapter, gently shake the device to insure particles are free flowing inside the chamber. To insert the Sylc™ SmarTip hold tube end with one hand and the handheld connector with the other and securely insert tube end into the connector plastic green collar.

**IMPORTANT –** After insertion pull on Sylc™ SmarTip to ensure secure engagement with the connector.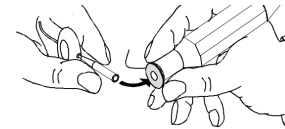


### STEP 3

Pull metal tip forward until stop position to unseal material prior to patient use.

**NOTE:** Turning metal tip prior to pulling reduces required pulling strength.

**IMPORTANT- Apply Sylc™ powder in a circular motion 3-4mm from the tooth surface. Apply for 1-5 seconds per tooth on entire dentition to remove stain and polish. Direct the tip toward the enamel at a 60° to 80° angle on buccal or lingual surfaces or at 90° to the occlusal surface. If tip rotates, position finger on tip to hold the tip in place during application.**

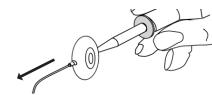Sylc™ SmarTip is now ready for use!

## TIPS FOR SUCCESSFUL APPLICATION

- For most efficient, comfortable operation and patient comfort, a pressure of 35-45 psi is recommended.
- Sylc™ SmarTip is a dry powder application. Patients may experience dry mouth. Rinse and evacuate after every quadrant for patient comfort.
- The angle of the tip varies with the position of the teeth and the surface to be cleaned and polished.
- The tip is held about 3-4 mm from the tooth surface; direct the tip toward the enamel at a 60° to 80° angle on buccal or lingual surfaces or at 90° to the occlusal surface.
- Use a small circular motion technique to clean and polish tooth surface. If tip rotates, positioning finger on tip may be required to hold the tip in place during application.
- Saliva ejector can be used in conjunction with high-speed evacuator.
- The high speed evacuator should be positioned as close to the metal tip as practical by operator or assistant.
- Patient will experience a gritty feeling in their mouth for a few minutes after treatment. This is expected because Sylc™ particulates are adhering to tooth surfaces to provide therapeutic benefits of desensitization.
- After the completion of treatment the patient should rinse well with water.
- It is recommended that the patient not drink or eat at least one hour after treatment.

**Sylc™ SmarTip Customer Service:  
866-238-9902**

#### Item# 95101001 - Starter Kit:

Sylc™ SmarTip Single Use Prophy Therapy Tips and Adapter (Quantity: 10 Prophy Tips & 1 Adapter)

#### Item# 95101501 - Refill Kit:

Sylc™ SmarTip Single Use Prophy Therapy Tips (Quantity: 25)

#### Item# 90100099 - Adapter:

1 Sylc™ SmarTip Adapter

**OSSPRAY™**  
Building Stronger Smiles. Naturally.

[www.osspray.com](http://www.osspray.com)

MANUFACTURED FOR: OSSpray Ltd, Hodgkin Building, King's College London, London, SE1 1UL, United Kingdom

OSSpray Inc, 301 Pleasant Street, Abbottstown, PA 17301

MANUFACTURED BY: GromanDental.com, 4900 NW 15th Street, Suite 44494, Margate, FL 33063  
Manufactured in USA.

Patent Numbers: US 7,329,126 & 7,607,972;  
EU 1,372,574.

Additional US and International Patents Pending.  
PM-0049 7/10

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HERGESTELLT FÜR: OSSpray Ltd, Hodgkin Building, King's College London, London, SE1 1UL, United Kingdom
OSSpray Inc, 301 Pleasant Street, Abbottstown, PA 17301, U.S.A