## DIRECTIONS FOR USE

POWERED BY NOVAMIN®

### Sylc™ SmarTip Single Use Prophy Therapy Tips and Pressure Regulated Adapter

Thank you for purchasing the Sylc™ SmarTip system. Please observe the following information to assure optimal use and safe operation.

**CAUTION:** Federal law restricts this device to be used by or on the order of a licensed dentist.

**INDICATIONS:** Sylc™ SmarTip is a single-use self-contained air and powder device that is powered by standard house air available at every dental operator. Sylc™ SmarTip is intended to be used with the Sylc™ SmarTip Pressure Regulated Adapter.

**DESCRIPTION:** Sylc™ SmarTip is an air polishing device prefilled with Sylc™ cleaning, polishing and desensitization powder designed to provide a prophylaxis treatment for dental enamel and to physically occlude dentin tubules for the management of sensitive teeth. Sylc™ SmarTip is a dry powder application, as such it is recommended to slightly elevate the patient's position and to rinse and evacuate after each quadrant for immediate cleaning results or wait until the end of the treatment. It is recommended that the patient not drink or eat at least an hour after treatment. Sylc™ SmarTip is marked with a pink dot.

**INGREDIENTS:** Sylc™ is a dry powder (calcium sodium phosphosilicate, Novamin®), composed of elements that occur naturally in the body's hard tissues (Ca, Na, Si, P, and O). When exposed to an aqueous environment, Sylc™ undergoes a rapid surface reaction, allowing it to physically adhere to exposed dentin and to physically occlude tubules. Within a short period of time, essentially all of the Sylc™ reacts to form hydroxycarbonate apatite (HCA), which is chemically similar to natural tooth mineral. Sylc™ has a neutral flavor.

**Sylc™ SmarTip removes plaque, soft deposits, and surface stains. It is also recommended for the following procedures:**

- Removal of mild to moderate extrinsic stains, e.g. coffee, tea and tobacco
- General cleaning and polishing during the prophylaxis treatment
- Treatment of hypersensitivity
- Surface preparation prior to bonding and sealants
- Before and after scaling and root planning procedure to clean surface and interproximal stains and desensitize
- Plaque removal prior to fluoride treatments
- Treatment of hypersensitivity pre and post whitening treatments

**CONTRAINDICATIONS:** Sylc™ should not be considered for patients with a known allergy to silica.

**DESCRIPTION:** Sylc™ SmarTip Pressure Regulated Adapter. The adapter screws directly to the standard chair side hand piece connector (compatible with 2,3,4,5 and 6 hole adapters). The SmarTip Adapter will operate at an air pressure range between 38-46 psi (2.6-3.2 bar). If the line air pressure is above 46 psi/3.2 bar, the adapter will reduce the operating pressure down to the preset range. Air flow is controlled by the standard chair side foot pedal.

**NOTE:** The metal tip has a small mark on its surface designed to regulate the powder flow and improve patient comfort.

**Sylc™ SmarTip Customer Service: 866-238-9902**

**Item# 95201001 - Starter Kit:** Sylc™ SmarTip Single Use Prophy Therapy Tips and Pressure Regulated Adapter (Quantity: 10 Prophy Tips & 1 Pressure Regulated Adapter)

**Item# 95201501 - Refill Kit:** Sylc™ SmarTip Single Use Prophy Therapy Tips (Quantity: 25)

**Item# 90200099 - Pressure Regulated Adapter:** 1 Sylc™ SmarTip Pressure Regulated Adapter

### CLEANING OF THE ADAPTER:

**DO NOT AUTOCLAVE** the SmarTip Pressure Regulated Adapter. PRIOR TO THE FIRST USE, AND AFTER EVERY PATIENT, thoroughly clean and disinfect the surfaces of the Adapter by applying an approved, non-immersion disinfectant solution. Carefully follow all of the directions provided by the manufacturer of the disinfectant. To clean the Adapter, generously spray the disinfectant solution onto a clean cloth and vigorously wipe down all surfaces. Immediately discard the used towel and allow the Adapter to dry after cleaning. It is recommended that the Adapter be sleeved for every use, using a standard plastic infection control sleeve for dental handpieces. It is also recommended to spray air into the opening of the Adapter to remove remaining powder from the chamber for optimal performance. Thoroughly clean and disinfect the surface of the Adapter by applying an approved, non-immersion disinfectant solution.

### STABILITY:

- Do not use if Sylc™ SmarTip is damaged.
- Store in a DRY place at room temperature, between 15-25 °C and 60-75 °F.

### HOW SUPPLIED:

- Sylc™ SmarTip is marked with a PINK dot.
- Sylc™ SmarTip is supplied as a starter kit containing 10 prefilled single use tips and one adapter.
- Sylc™ SmarTip is supplied as a refill kit containing 25 prefilled single use tips.

**WARRANTY:** Adapter has a 2 year warranty, subject to change without notice. Autoclaving or misuse of the Adapter will void warranty terms. Warranty terms do not apply on the prefilled single use tips.

**TECHNICAL CHANGES:** OSSpray reserves the right to make changes to Sylc™ usage instructions when required in the course of ongoing technical and scientific developments.

### SAFETY PRECAUTIONS:

Read the instructions carefully before using this device. The manufacturer, distributor, or retailer of this product can exercise no control over the use of the Sylc™ SmarTip and adapter.

- The Sylc™ SmarTip is a single use device. Dispose after single patient use regardless of the remaining quantity of powder.
- During the preparation process and application, it is recommended that both clinician and patient wear protective eyewear. If any material gets into the eyes, immediately flush with water.
- Delivering Sylc™ directly at the gum tissues may cause injury to the gum tissues or an emphysema caused by the introduction of air into the soft tissue spaces.
- It is strongly recommended not to direct the metal tip / Sylc™ powder at the gum tissue or into the gingival sulcus.
- Avoid directing Sylc™ powder toward restorations, crowns and bridgework as this could damage the restoration.
- Avoid directing Sylc™ powder on orthodontic brackets as this could dull the metal.
- DO NOT attempt to sterilize, refill or reuse.
- Mounting and pressurizing the device more than once will degrade device operation.
- Clinicians may practice and experiment on typodonts, pennies, stainless steel, extracted teeth, or glass slides before actual use. These surfaces will simulate both precious and non-precious alloys and porcelain.
- Safety and effectiveness in pregnant women and children have not been established.
- Federal Law restricts this device to sale by or on the order of a licensed dentist.

**OSSPRAY™**  
Building Stronger Smiles. Naturally.

[www.osspray.com](http://www.osspray.com)

OSSpray Ltd, Hodgkin Building,  
King's College London  
London, SE1 1UL  
United Kingdom

OSSpray Inc, 301 Pleasant Street  
Abbottstown, PA 17301

Patent Numbers: US 7,329,126 & 7,607,972; EU 1,372,574.

Additional US and International Patents Pending.

Manufactured by GromanDental.com, 4900 NW 15th Street, Suite 4494, Margate, FL 33063


Sylc™ is a registered trademark of OSSpray, Ltd. Novamin is a registered trademark of GlaxoSmithKline plc.

## USER INSTRUCTIONS

**⚠ IMPORTANT - REVIEW SAFETY PRECAUTIONS PRIOR TO USE. Manufacturer is not responsible for damages due to misuse of the product.**


### STEP 1

Attach the adapter to a low speed air hose.  
**NOTE:** Water delivery port is sealed off.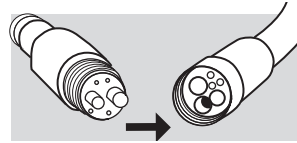


### STEP 2

Gently shake the blister tip to ensure the particles inside are free flowing.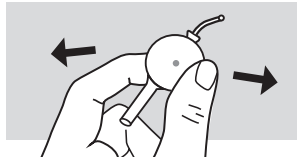

### STEP 3

Pull back on the plastic collar, insert the blister tip and release collar.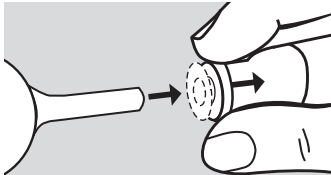

## TIPS FOR SUCCESSFUL APPLICATION

- Sylc™ SmarTip is a dry powder application. Slightly elevate patient's position in the chair for increased comfort. Patients may experience dry mouth. Rinse and evacuate after every quadrant for patient comfort.
- Heavy calculus build-up may effect cleaning and desensitization results.
- The tip is held about 3-4 mm from the tooth surface; direct the tip toward the enamel at a 60° to 80° angle on buccal or lingual surfaces or at 90° to the occlusal surface.
- Use a small circular motion technique to clean and polish tooth surface. If tip rotates, positioning finger on tip may be required to hold the tip in place during application.


### STEP 4

Gently pull on the blister tip to ensure it is seated securely in the adapter.  
**NOTE:** DO NOT TWIST BLISTER TIP WHILE SEATED IN ADAPTER.

### STEP 5

Extend metal tip forward until stop position to unseal material prior to patient use.

**WARNING:** Do not extend metal tip forward while the Adapter is in the upright position at chairside. This will release powder into the chamber and may result in suboptimal performance.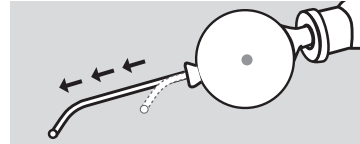



### STEP 6

Apply Sylc™ powder in a circular motion 3- 4mm from the tooth surface. Apply for 1-5 seconds per tooth on entire dentition to remove stain and polish. Direct the tip toward the enamel at a 60° to 80° angle on buccal or lingual surfaces or at 90° to the occlusal surface. If tip rotates, position finger on tip to hold the tip in place during application.

### Removal / Storage

Remove blister tip from adapter by pulling back on plastic collar (1) and at the same time pulling the blister tip away from the adapter (2). **DO NOT TWIST BLISTER TIP WHILE SEATED IN ADAPTER, THIS MAY CAUSE IT TO TEAR OFF AND REMAIN INSIDE ADAPTER RESULTING IN ADAPTER FAILURE.** Discard blister tip in sharps container.

**DO NOT AUTOCLAVE** the SmarTip Pressure Regulated Adapter. Autoclaving will void the warranty.

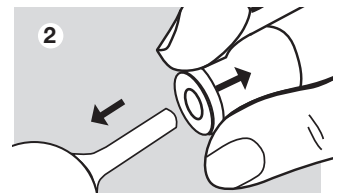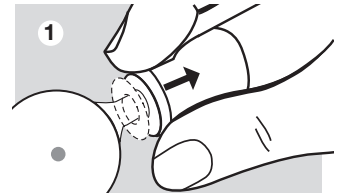Adapter may be stored at chair side or in a secure place in drawer.

- Saliva ejector can be used in conjunction with high-speed evacuator.
- The high speed evacuator should be positioned as close to the metal tip as practical by operator or assistant.
- Patient will experience a gritty feeling in their mouth for a few minutes after treatment. This is expected because Sylc™ particulates are adhering to tooth surfaces to provide therapeutic benefits of desensitization.
- After the completion of treatment the patient should rinse well with water.
- It is recommended that the patient not drink or eat at least one hour after treatment.

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